

ORTHO
SYSTEMS

2000
2026
Years in motion



Excellence in Neuro-Orthopedics


Your Patient – Our Contribution

Catalogue
Excerpt

Orthotics 14.0
Valid from July 2026


Standing Podiums in Comparison

Custom-Fabricated Body Shell: Precise support and alignment of the foot and ankle joints are essential for correct positioning of the knee and hip joints. Individually tailored to each user, the body shell addresses primary deformities using the three-point pressure principle, thereby proactively countering potential secondary deformities. Custom alignment, designed to meet the specific needs of the patient, ensures correct positioning of the head.

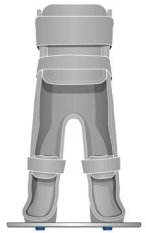


Stehbrett

Standard Model

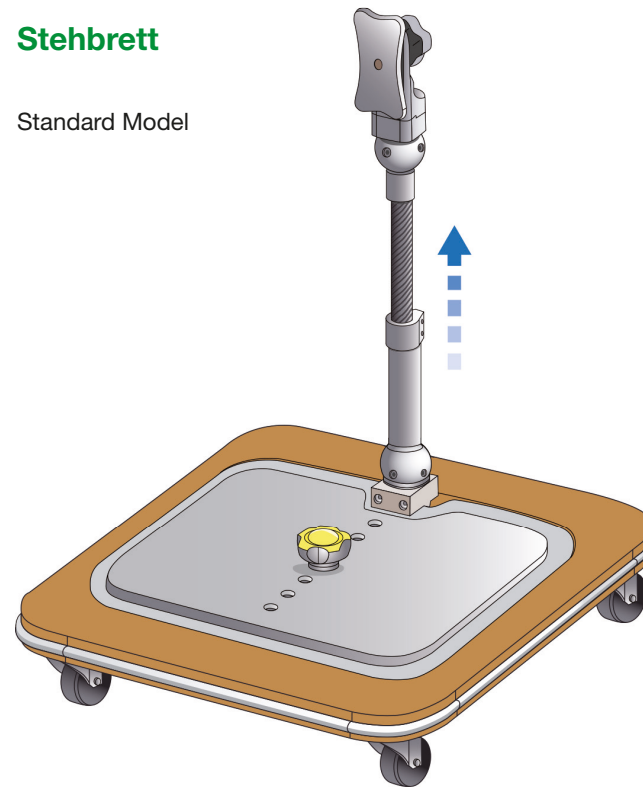

Stehfeld

Also for therapeutic use

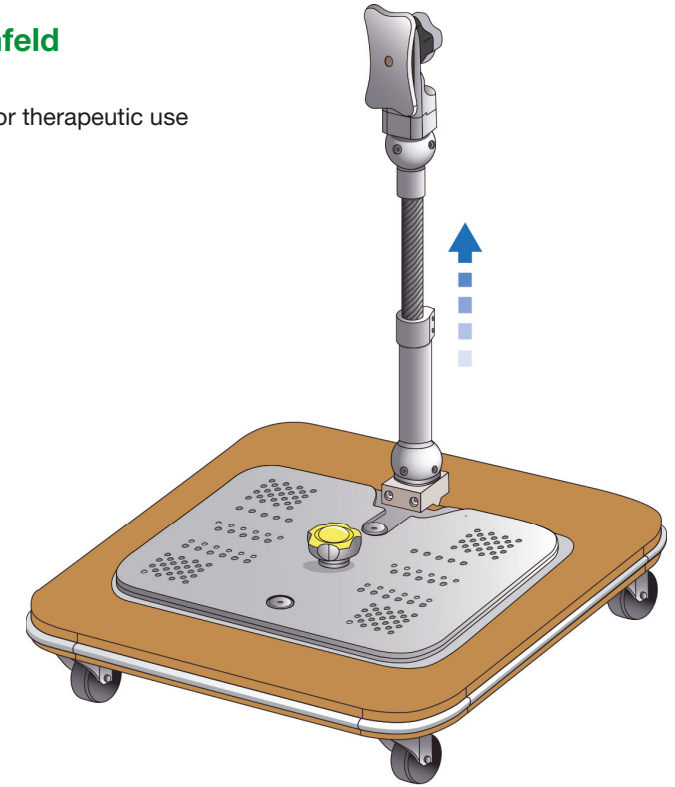


The standing platform with body shell: Children need to experience standing as soon as they attempt to upright themselves. It is important they perceive the world from a vertical perspective and can align their gaze horizontally. Equally important is that they are at eye level with other children. This enhances their well-being and level of attention. Furthermore, standing has positive effects on circulation and the cardiovascular system, respiration, digestion, kidney, and bladder function. It increases bone density, reduces the risk of fractures, and promotes the development of the hip joint. Standing also enables playing and working at a table.

Functional Overview Standing Podiums

Podiums are combined with an individual, custom-fabricated body shell to form the standing unit. The body shell is affixed to a universal connecting plate, which is fully interchangeable with all Stehbrett sizes, as well as Foot-Walker and Leicht-Walker podiums (see “Walking Podiums”). The Stehfeld podium comes with an adjustable, tilting connecting plate, which can be used for posture control exercises to augment therapy sessions. Hand grips and anti-tip options are available which can be added at any time and are compatible with all podiums.



			Body Shell			Modular Accessories		
Growable	Stability	Protective Bumper	3-Point Fixation	Removable	Posture Control Exercises	Hand Grips	Anti-tip Protection	
								 Stehbrett Pages 200 -201
								 Stehbrett-KS Page 201
								 Stehbrett-HG Page 201
								 Stehbrett-HG-KS Page 201
								 Stehfeld Page 202-203
								 Stehfeld-KS Page 203
								 Stehfeld-HG Page 203
								 Stehfeld-HG-KS Page 203

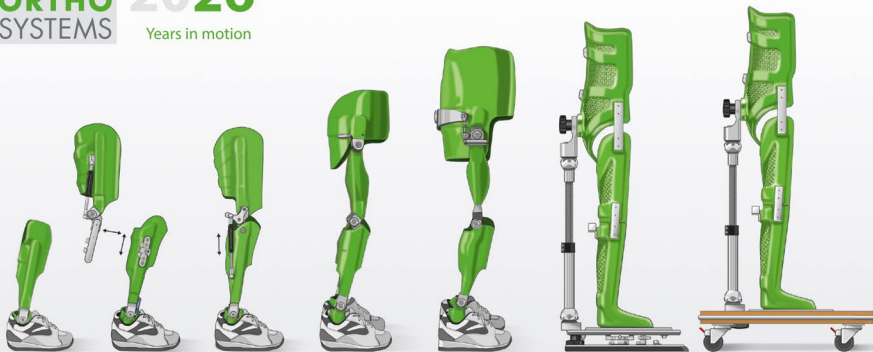

Contact

You can reach us from
by telephone
per Fax
per E-Mail
Address
Contact persons

9 am to 5 pm (Germany)
+49 89 904 7575 0
+49 89 904 7575 5
kontakt@ortho-systems-mail.de
Merowingerstr. 14, 85551 Kirchheim, Germany
Jeff Matwichuk and Thomas Böckh

www.ortho-systems-inmotion.com